



EZHRI™

V series Quick Guide



EzHRI™

1. Start EzHRI™

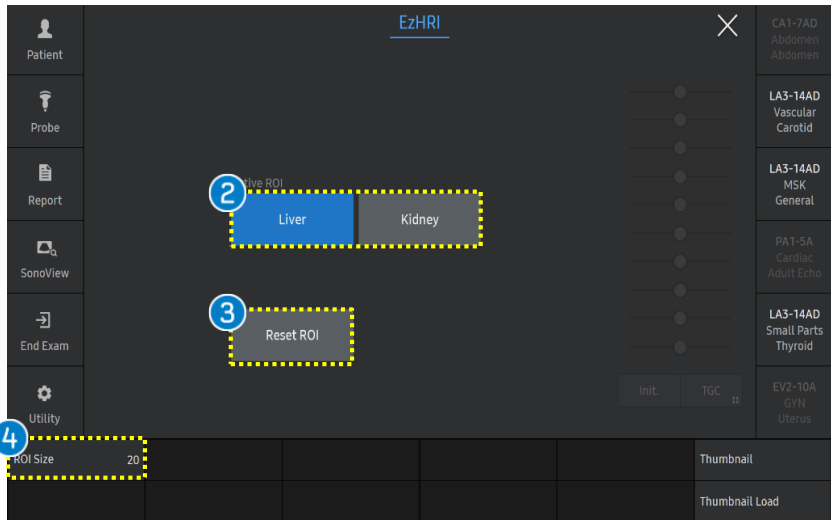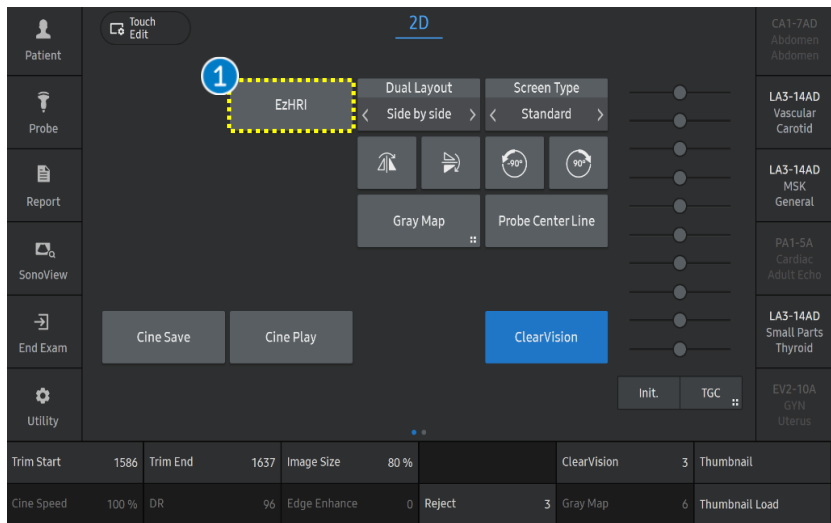

Acquire longitudinal hepato renal view of kidney and liver at the same screen.

Try to avoid shadowing and motion artifacts during the scanning.

1 EzHRI	Activate the [EzHRI] on frozen 2D mode.
2 Active ROI	To change ROI position and size, choose the desired ROI using the [Active ROI] button or the [Change] key.
3 Reset ROI	Tap the [Reset ROI] button on the touch screen. Then all ROI will be restored initial position.
4 ROI Size	It could change ROI size.
5 Set	Confirm the result value by pressing the [Set] button on the control panel. Then, the result value is saved and assigned to the report.

EzHRI™

2. The result of EzHRI™



ID	Name				
Date of Birth(Age)	Gender	Exam Date	2023-05-19		
Indication					
Diag. Physician	Ref. Physician	Operator			
Abdomen					
Height	Weight				
EzHRI					
	1	2	3	4	5
Liver	101.45	87.55	101.09	83.32	110.12
Kidney	102.40	93.68	81.21	63.42	89.32
HRI (L/K)	0.99	0.93	1.24	1.31	1.23

2

1 Result

- The result

Liver	121.20
Kidney	89.31
HRI (L/K)	1.36

Liver	The mean brightness level within the ROI on Liver
Kidney	The mean brightness level within the ROI on Kidney
HRI (L/K)	The ratio of the mean brightness levels of the liver and renal parenchyma $\text{HRI (Hepato renal index)} = \frac{\text{mean brightness of liver}}{\text{mean brightness of kidney}}$

2 Report

Select [Report] button on the touch screen and the result value is saved on the report.

- The features, options may not be commercially available in some countries.
- Sales and shipments are effective only after the approval by the regulatory affairs. Please contact your local sales representative for further details.
- This Quick guide does not include all of the details of instruction, for more detail, please refer to V series User Manual.
- Do not distribute this document to customers unless relevant regulatory and legal affairs officers approve such distribution.
- This User Quick Guide is based on V series V1.05.
- Disclaimer: Some Images in this content were obtained from other system.

SAMSUNG MEDISON CO., LTD.

© 2024 Samsung Medison All Rights Reserved.

Samsung Medison reserves the right to modify the design, packaging, specifications, and features shown herein, without prior notice or obligation.