

FAQ

Modifying Image Parameters for EzStructure™


[FAQ – EzStructure™]

- **Question**
Can I change the image parameters for **EzStructure™**?

- **Answer**

■ Path: Setup > Imaging > EzStructure

1. Select **EzStructure** list.
(Select **Rename** to edit names.)
2. Select a **Probe**, **Application**, and **Preset**.
3. Adjust the image parameters in 2D Mode.

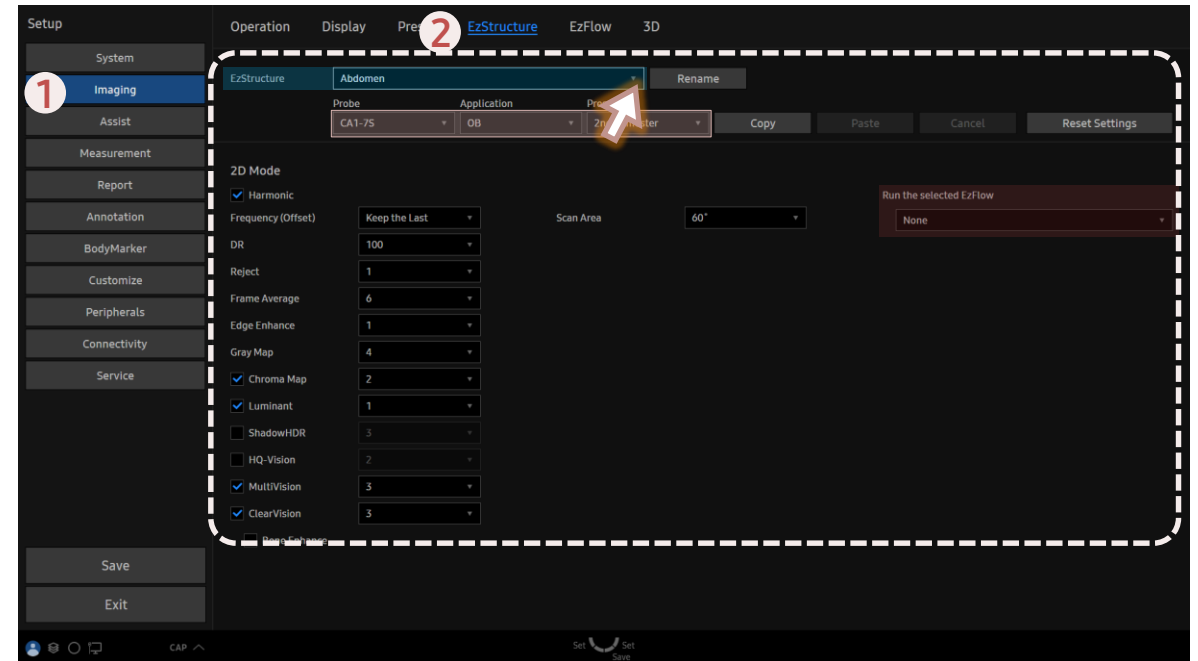

- When ticked, the corresponding function is activated, and you can set the index.
- While using **EzStructure**, select **EzFlow** as the default option for Color mode entry.