


FAQ

Sending 3D volume data format to PACS

[FAQ – Sending 3D volume data format to PACS]

- **Situation**
I want to send data in a 3D volume data format instead of a 3D image.
- **Solution**
 - **Path:** Utility > Setup > Connectivity > DICOM tab
 - Select the **Storage** from the service items.
 - Click **Edit**.
 - Check the **Include 3D volume** checkbox.

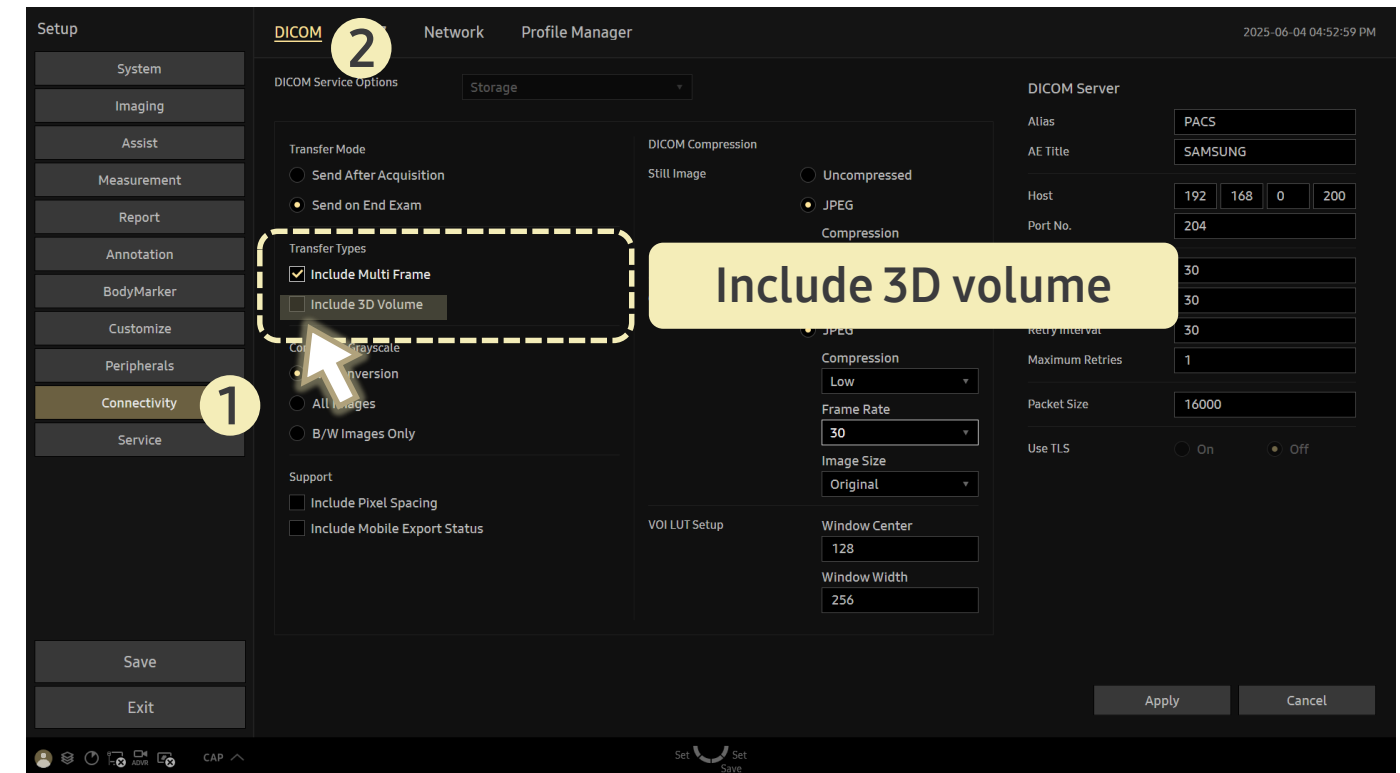

* When 3D images are sent, 3D volume data are also sent.
Select this option only if you use a storage service that supports our 3D volume data format.