

**FAQ**  
**DICOM Compression**  
**- Cine 1 -**

## [FAQ – DICOM Compression (Cine) 1]

- **Situation**  
I want to reduce the file size of DICOM Cine images sent to PACS.
- **Solution**
  - **Path: Utility > Setup > Connectivity > DICOM tab**
  - Select the **Storage** from the service items.
  - Click **Edit**.
  - Set the degree of **Compression**.

\* DICOM Compression settings are applied at the time an image is saved.  
If it is not transmitted to PACS, it means that the PACS does not support compression. Change it to Uncompressed.

