

**FAQ**  
**DICOM Compression**  
**- Still Image -**

## [FAQ – DICOM Compression (Still Image)]

- Situation  
I want to reduce the file size of DICOM Images sent to PACS.
- Solution
  - Path: Utility > Setup > Connectivity > DICOM tab
  - Select the **Storage** from the service items.
  - Click **Edit**.
  - Set the degree of **Compression**.



\* You can select Uncompressed (Save images without compression) or JPEG (Compress and store images).  
DICOM Compression settings are applied at the time an image is saved.