

## FAQ

# [Storage] Configuring Selective PACS Transmission for User Keys

## [FAQ – Configuring Selective PACS Transmission for User Keys]

- Situation

I want to assign the PACS transmission function to only some User Keys, while configuring other keys to save images only on the ultrasound system without transmitting them to PACS.

- Solution

1. Go to [Utility](#) > [Setup](#) > [Connectivity](#) > [DICOM Tab](#) > [Storage](#), and set the Transfer mode to [Send on End Exam](#).
2. Disable the [DICOM Destination](#) option on the Patient screen.
3. Navigate to [Utility](#) > [Setup](#) > [Customize](#) > [User Key Tab](#), activate the [Send](#) option, and assign it to the respective PACS server.