

FAQ

[Storage] Transfer mode
- Send After Acquisition -

[Storage] – Transfer mode: Send After Acquisition

Transmit Behavior	Send After Acquisition	User Key (P1/2/3) (Setup – Customize)	Destination (Patient screen)
No images are transmitted.	Enabled	Disabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Disabled
Transmit only the images saved when the User Key is pressed. - Excludes transmission of images not saved via the User Key (e.g., “Save Patient Page as First Image” or touchscreen saves)	Enabled	Enabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Disabled
Transmits each time a thumbnail is generated on the right side of the monitor, including “Save Patient Page as First Image.”	Enabled	Disabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Enabled
Images are sent simultaneously to both the storage defined in the User Key settings and the configured destination each time the User Key is pressed. ※ Duplicate images may appear once or multiple times in the PACS Viewer program, depending on the PACS configuration.	Enabled	Enabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Enabled