

FAQ

[Storage] Transfer mode
- Send on End Exam -

SAMSUNG R20

[Storage] – Transfer mode: Send on End Exam

Transmit Behavior	End Exam	User Key (P1/2/3) (Setup – Customize)	Destination (Patient screen)
No images are transmitted.	Enabled	Disabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Disabled
Only images transmitted when User Key is pressed. - Excludes transmission of images not saved via the User Key. (e.g., "Save Patient Page as First Image" or touchscreen saves)	Enabled	Enabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Disabled
All thumbnail images transmitted upon End Exam. - Includes "Save Patient Page as First Image". - No images are transmitted via the User Key.	Enabled	Disabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Enabled
Images are transmitted to the storage specified in the User Key settings each time the User Key is pressed. Additionally, at the end of the exam, all thumbnail images are resent to the configured destination. ※ Duplicate images may appear once or multiple times in the PACS Viewer, depending on its configuration.	Enabled	Enabled <input checked="" type="checkbox"/> Store <input checked="" type="checkbox"/> Send	Enabled

► Configure only the **DICOM Destination**. If "Send" is enabled in the **User Key**, images will be transmitted every time the User Key is pressed.