

FAQ

How to Open Search Screen Directly from SonoView


[FAQ – End Exam (SonoView)]

- Question
When I tap **SonoView** on the touchscreen, it shows the review screen with previous exams. I want it to open the **Search** screen directly. How can I set this?

- Answer

■ Path: Setup > Customize > General

1. Check the box for **Clear Exam List in SonoView**.
2. This automatically clears the list of open exams after ending an exam, so **SonoView** opens directly to the **Search** screen.

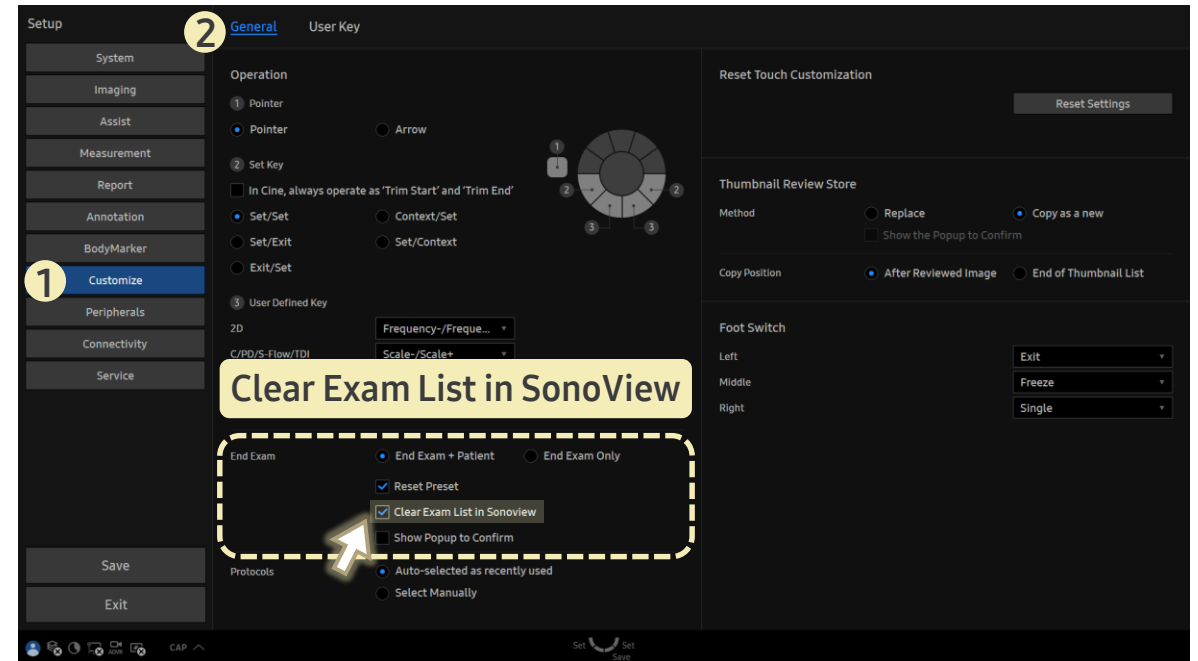

- If this option is unchecked, **SonoView** will first show the previously opened exams for review. To access the **Search** screen, you will need to click **Search Exam** manually.
* **SonoView** displays the list of exams currently under review.