


FAQ

How to Select EFW Method and Change Measurement Sequence

[FAQ – EFW Author and Sequence]

- Question
Where can I set the **Author** and **Measurement Sequence** for **EFW**(Estimated Fetal Weight)?
- Answer
 - Path: Setup > Measurement > Author
 - 1. Select the author in **EFW1**.
 - 2. In **EFW1 Measure Sequence**, choose the desired biometry measurement items and adjust the order using the up/down arrows.

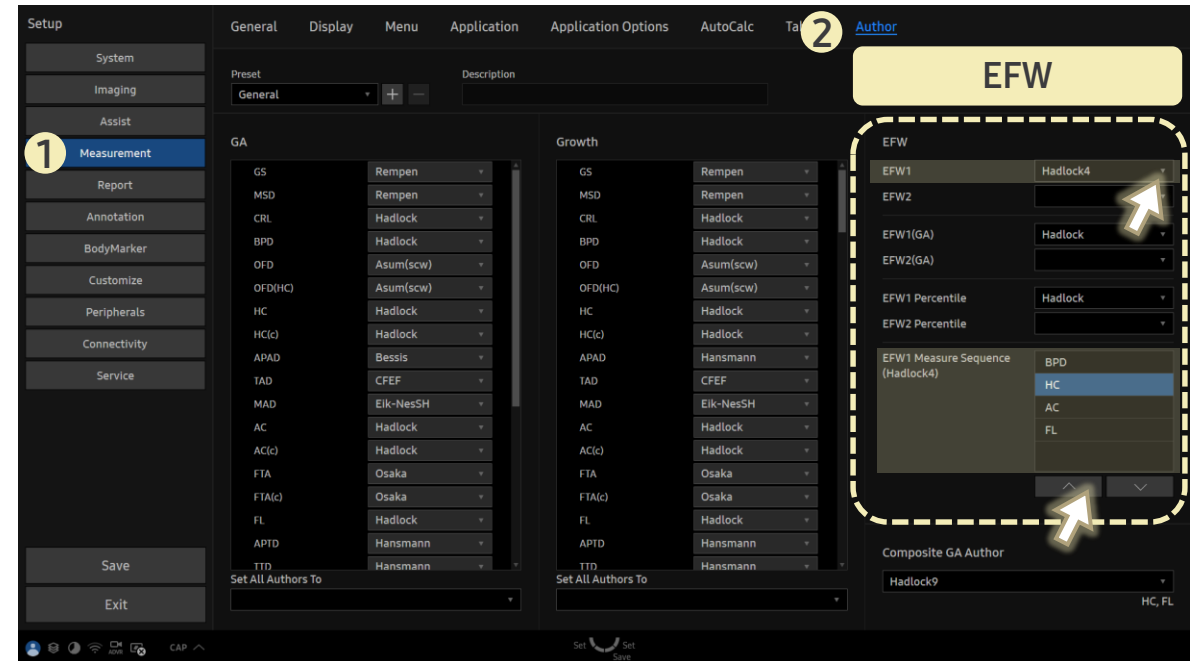

- **EFW2** is used to compare different authors for **EFW1**.