

## FAQ

# How to Set Text Options When Reloading Thumbnail Images and Performing Live Scans


## [FAQ – Annotation]

- Question  
After reviewing a thumbnail image saved with text, I switched to live scan mode. Can I configure the settings to retain the text from the saved image?

- Answer

■ Path: Setup > Annotation > General

1. Enable **Keep the Text in Live Scan after Reloading** by checking the checkbox.
2. Select **Text in Reloaded Image** to retain the text from the reloaded thumbnail image.

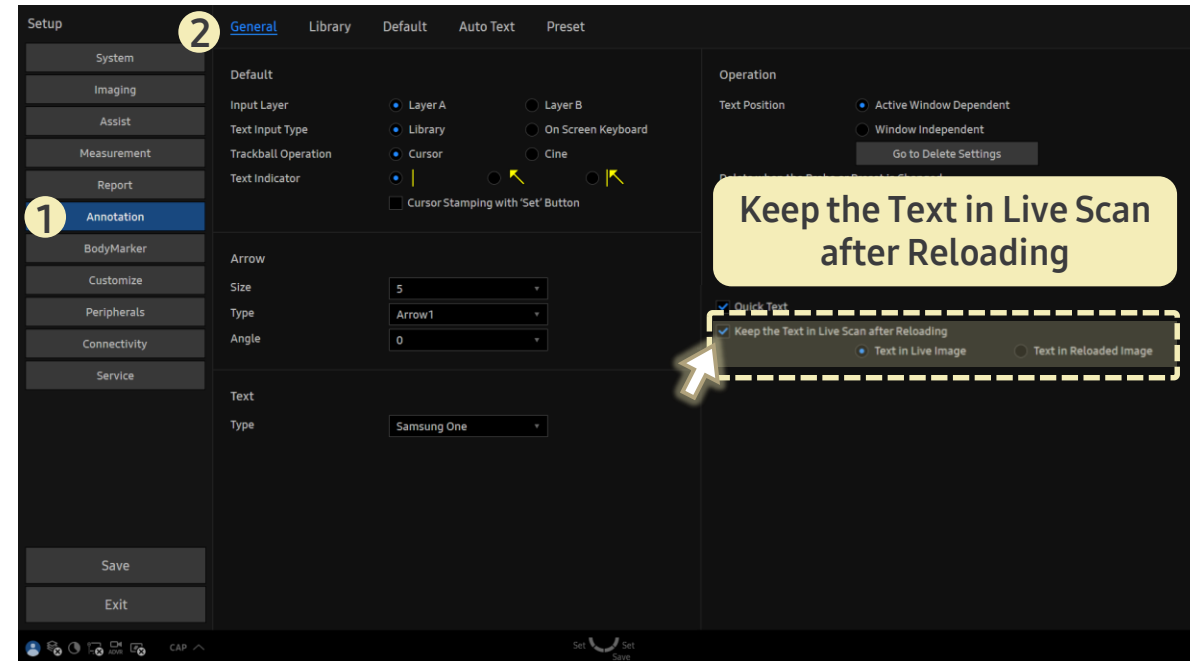

When **Text in Live Image** is selected, the text entered in live scan mode will be retained.

\* However, if the settings in **Setup > Annotation > Preset** are configured to delete text upon unfreeze, the text cannot be retained.