

FAQ
How to Modify
EzFlow™ Image Parameters


[FAQ – EzFlow™]

- Question
Can I customize **EzFlow™** image settings for a specific application?

- Answer

■ Path: Setup > Imaging > EzFlow

1. In the **EzFlow** dropdown, select the desired **Application** and corresponding **EzFlow item**.
2. Select the target **Probe** and **Preset** for customization
3. Adjust image parameters for Color, Power Doppler, PW, and CW modes.

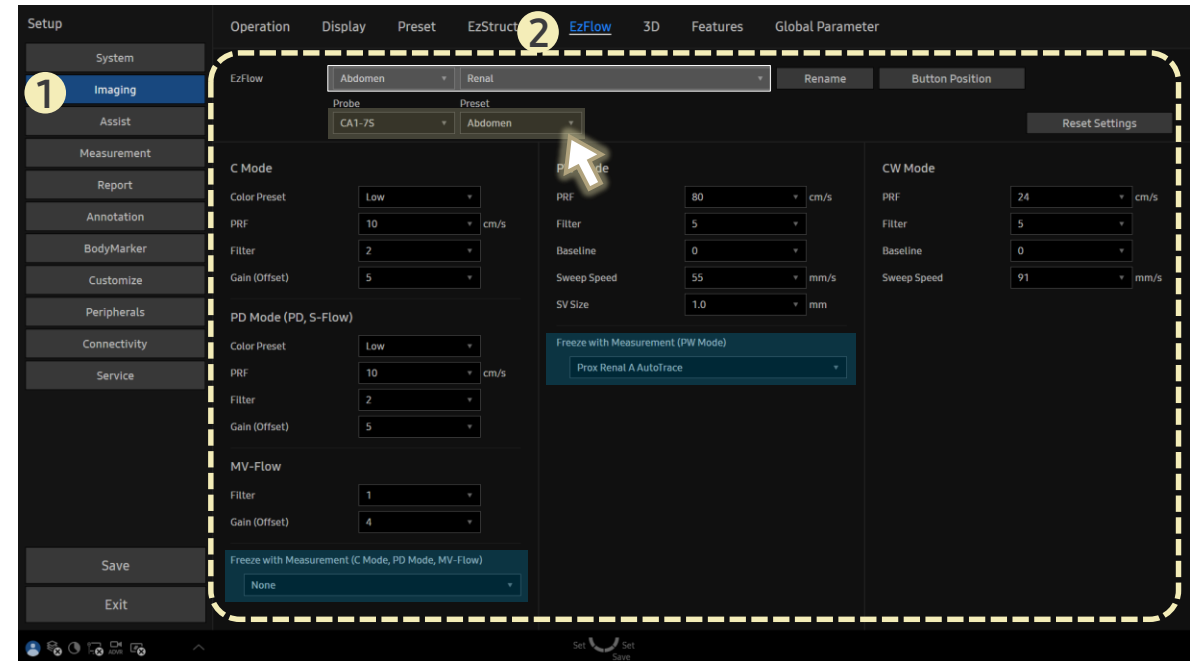

Freeze with Measurement

- Setup> Imaging> Preset settings have priority.
- Setup> Imaging> EzFlow, you can individually set the measurement after Freeze for each EzFlow item.