

## FAQ

# How to Enable ViewAssist™ for Auto Measure and Annotation in OB Scans


## [FAQ – ViewAssist™]

- Question  
How can I activate automatic measurement and annotation in OB scans?

- Answer

■ Path: Setup > Assist > OB > General

1. Make sure **ViewAssist** is set to **On**.
  - ▶ This enables the system to recognize and classify fetal biometry and fetal heart images automatically, which is required for using **Auto Measure** and **Auto Annotation**.

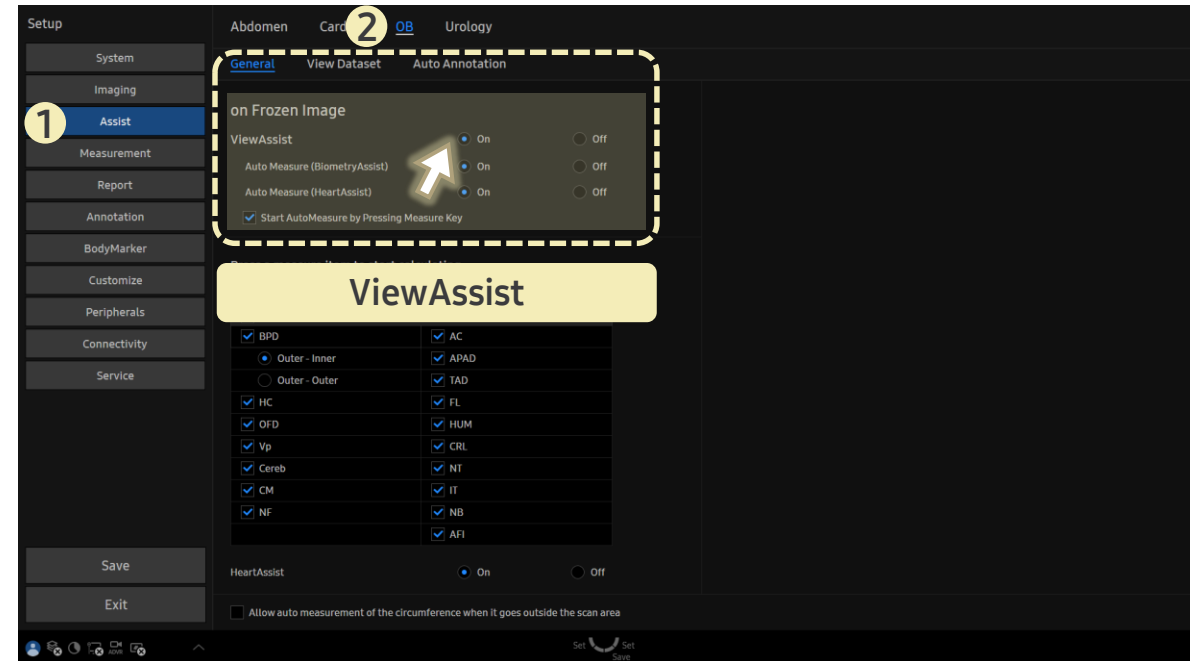

After pressing **Freeze** or when reviewing saved images, you can find **Auto Annotation** and **Auto Measure** on the touchscreen in

- **Assist** section
- **User** section: Need to add the two items at **Utility > Touch Customization > Special > User**