

FAQ

How to Enable Z-Scores in Pediatric Measurements

[FAQ – Z-Scores in Pediatric exams]

- Question
I want to use **Z-scores** in Pediatric exams. How can I enable this feature?
- Answer
 - Path: Setup > Measurement > Application Options
 - 1. From the **Application** dropdown, select **Pediatric**.
 - 2. Set **Z-scores** to **On**.



- Z-scores are available for measurements including Kidney (length, volume), Liver (length), Spleen (length), Uterus (volume), Ovary (volume), Testis (volume), etc.