

FAQ

How to Customize the Display Order of Vascular Measurement Names


[FAQ – Vascular Naming]

- Question
Can I change the order of how laterality, location, and name are displayed in vascular measurements?

- Answer

■ Path: Setup > Measurement > General

1. Go to **Vascular Naming**.
2. Choose a preferred naming order from the dropdown.

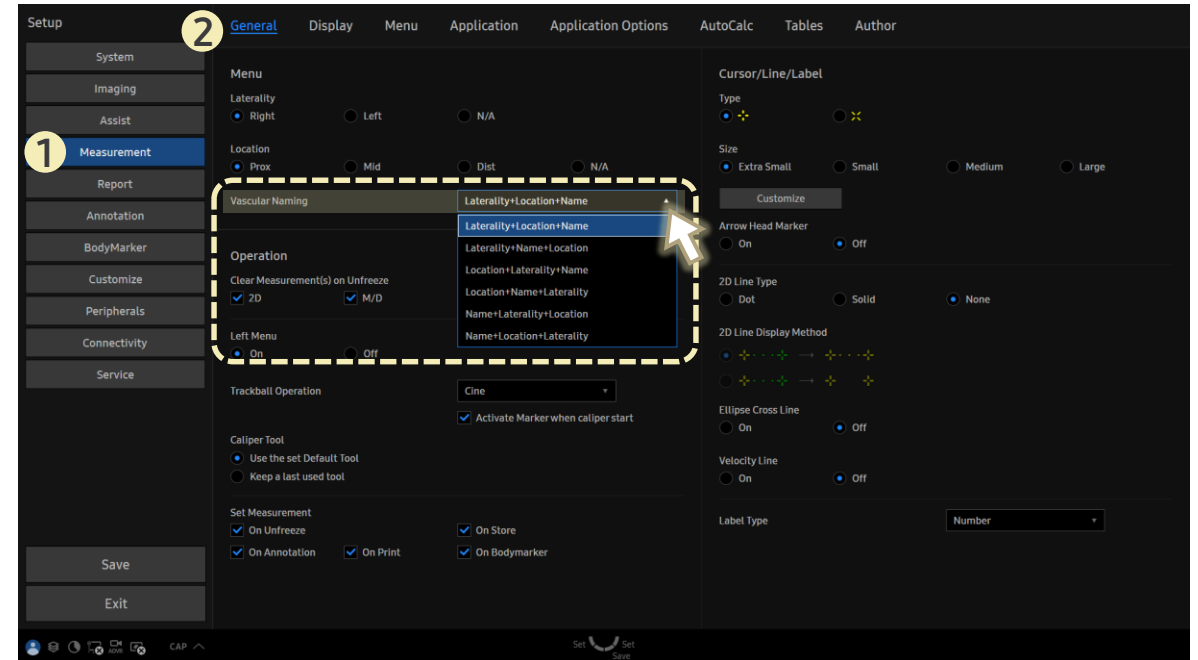

Naming Order Options

- Laterality + Location + Name / Laterality + Name + Location
- Location + Laterality + Name / Location + Name + Laterality
- Name + Laterality + Location / Name + Location + Laterality