

FAQ

How to Delete or Retain Measurement Results on Unfreeze

[FAQ – Measurement Results]

• Question


I want to delete or retain measurement results when unfreezing. I set this up in **Setup > Measurement > General** tab under **Clear Measurement(s) on Unfreeze** option. However, the settings are not being applied. Where should I change the settings?

• Answer

■ Path: Setup > Measurement > Application Options

The **General** tab allows you to configure settings for the entire application.

To configure individual applications, go to the **Application Options** tab. Use the checkbox for **Clear Measurement(s) on Unfreeze** to set it for each specific application as needed.

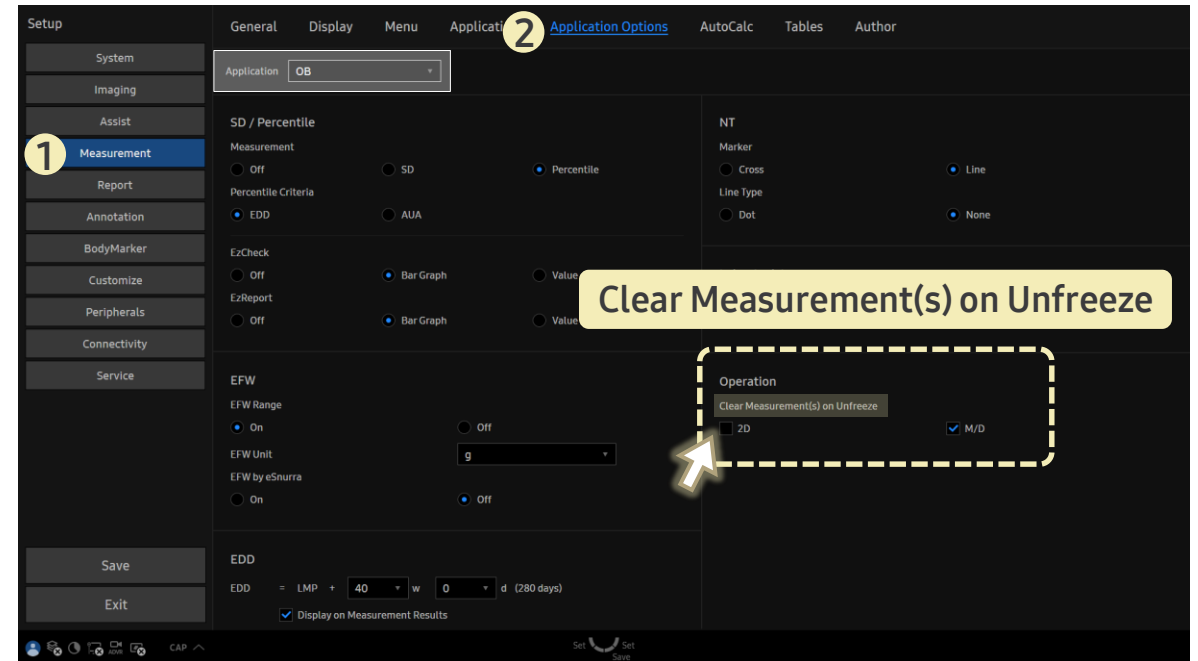

- Checked: Measurement results are deleted upon unfreeze.
- Unchecked: Measurement results remain on the screen and are displayed cumulatively with additional measurements.