


## FAQ

# How to Display or Hide Patient Information During Scanning

# SAMSUNG R20

## [FAQ – Patient Information]

- Question  
Can I set the system so that patient information is hidden on the scan screen during scanning but visible when saving or sending to PACS?
- Answer
  - Path: Physical keyboard > Patient Info.
  - 1. Pull out the physical keyboard(\* Option) located at the bottom of the control panel.
  - 2. Press **Patient Info.** Key  (located to the right of the **ESC** key, above the **number 3** key).



- Patient information remains hidden during scanning, but this setting does not affect saved images or DICOM transmission.
- Each press of **the Patient Info.** key turns the patient information on or off.