


FAQ

How to Configure the Default Annotation Library

[FAQ – Default Annotation Library]

- Question
Where can I configure the default **Annotation Library** that appears on the touchscreen in **Text** mode?
- Answer
 - Path: Setup > Annotation > Default
 - 1. Select **Probe**, **Application**, and **Preset**.
 - 2. Select the desired item from the **Library List**.
 - 3. Choose the desired position from **Touch Screen Tab** to assign it.
 - 4. Click the circular button above to set the library as default.

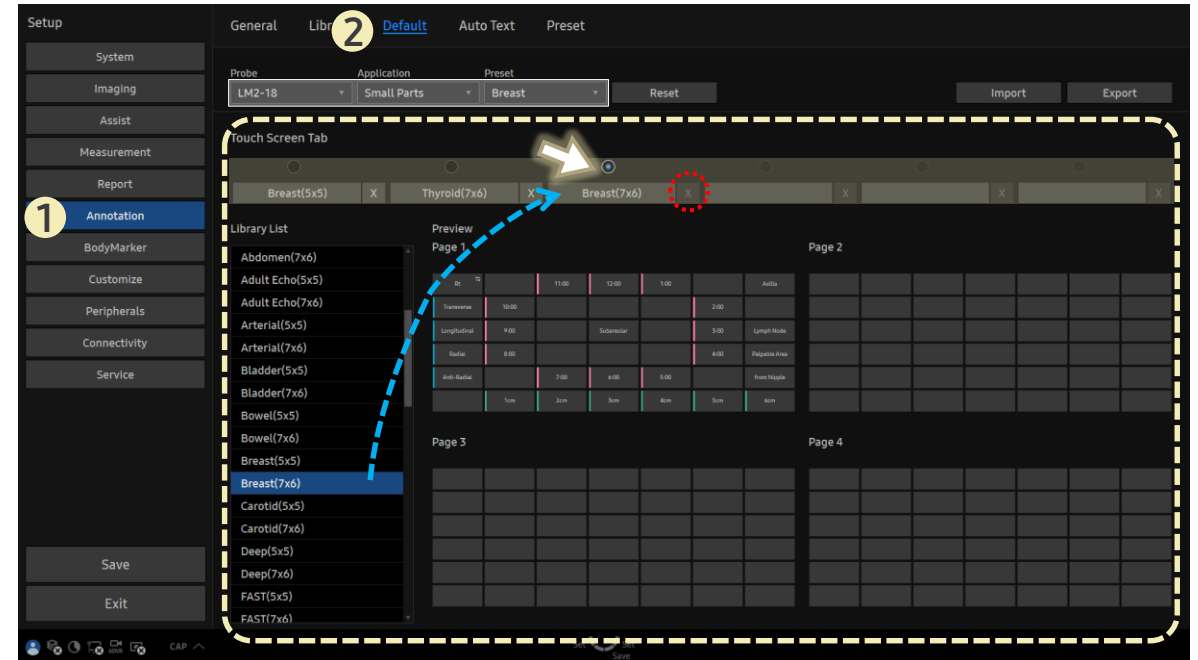

- To delete a library assigned to a touchscreen tab,
- Click the **X** button next to it.