

## FAQ

# How to Configure the Default BodyMarker Library


## [FAQ – Default BodyMarker Library]

- Question  
Where can I configure the default **BodyMarker Library** that appears on the touchscreen in **BodyMarker** mode?

- Answer

■ Path: Setup > BodyMarker > Default

1. Select **Probe**, **Application**, and **Preset**.
2. Select the desired item from the **Library List**.
3. Choose the desired position on the **Touch Screen Tab** to assign it.
4. Click the circular button above to set the library as default.

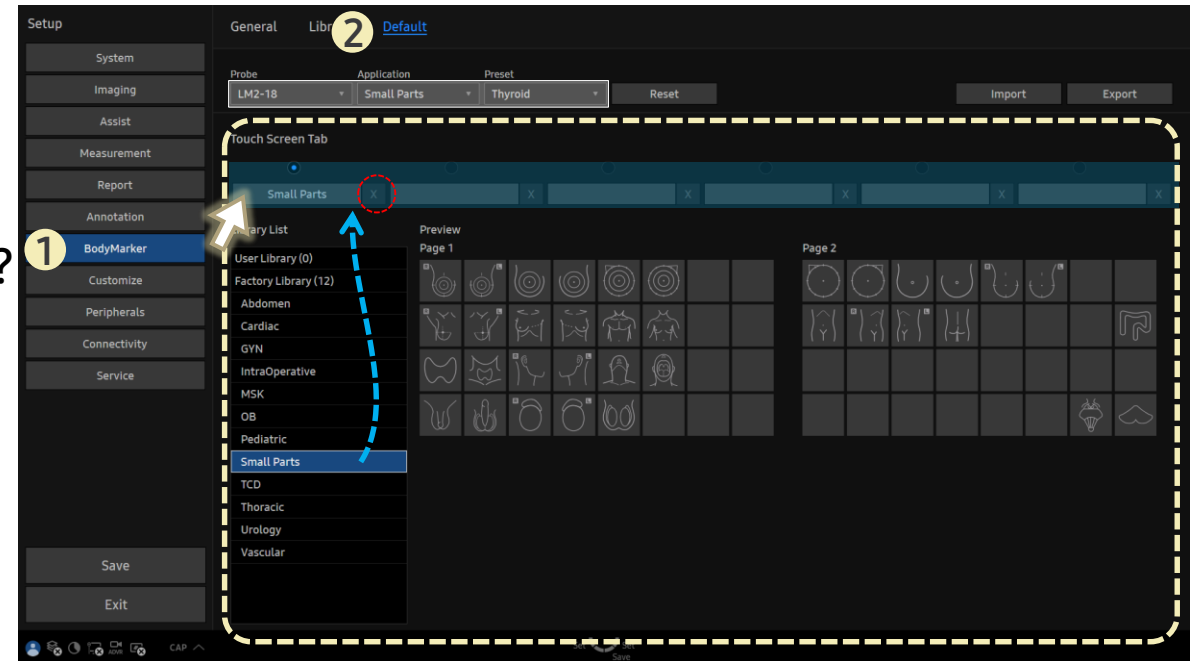

- To delete a library assigned to a touchscreen tab,
- Click the **X** button next to it.