

FAQ

How to Change the C/PW Acquisition Speed

[FAQ – C/PW Acquisition Speed]

- Question
How can I change the speed of positioning the ROI/SV in C/PW mode?

- Answer

■ Path: Setup > Imaging > Operation

Select the speed of the **C/PW Acquisition Speed after Positioning ROI/SV**.

- ▶ Sets the speed of positioning the ROI/SV in C/PW mode.



- Changes the speed of positioning the **ROI** in **C-mode** and **SV** in **PW-mode**.