

FAQ

How to Change the Increments of the Values Adjusting Image Parameters


[FAQ – Knob Control Speed]

- Question
Can I change the increments of the values adjusting the imaging parameters?

- Answer

■ Path: Setup > Imaging > Operation

1. Go to **Knob Control Interval** settings.
2. Set the value increments to adjust **Gain**, **DR** (Dynamic Range), and **Zoom ROI Size** from **Slow** to **Fast**.
 - ▶ The selected increment applies to both the 2D and 4D mode.

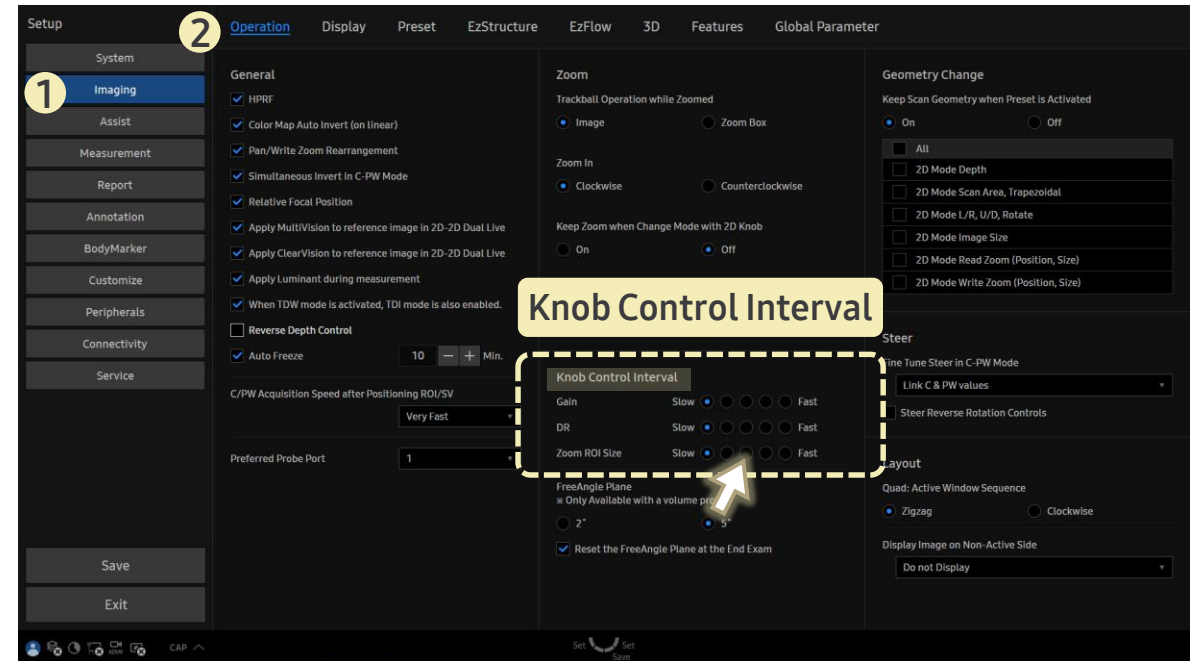

- You can configure the increments for adjusting the image parameters by selecting the desired knob button and speed under **Knob Control Interval**.
- This determines the change of values by the adjusted increments when the button is turned.