

FAQ

How to Assign Actions When Ending an Exam


[FAQ – End Exam]

- Question
How can I assign actions when **Ending an Exam**?

- Answer

■ Path: Setup > Customize > General

- Select **End Exam + Patient** to end the exam and immediately access the new patient registration screen.
- Select **End Exam Only** to end the exam while keeping scan mode active.

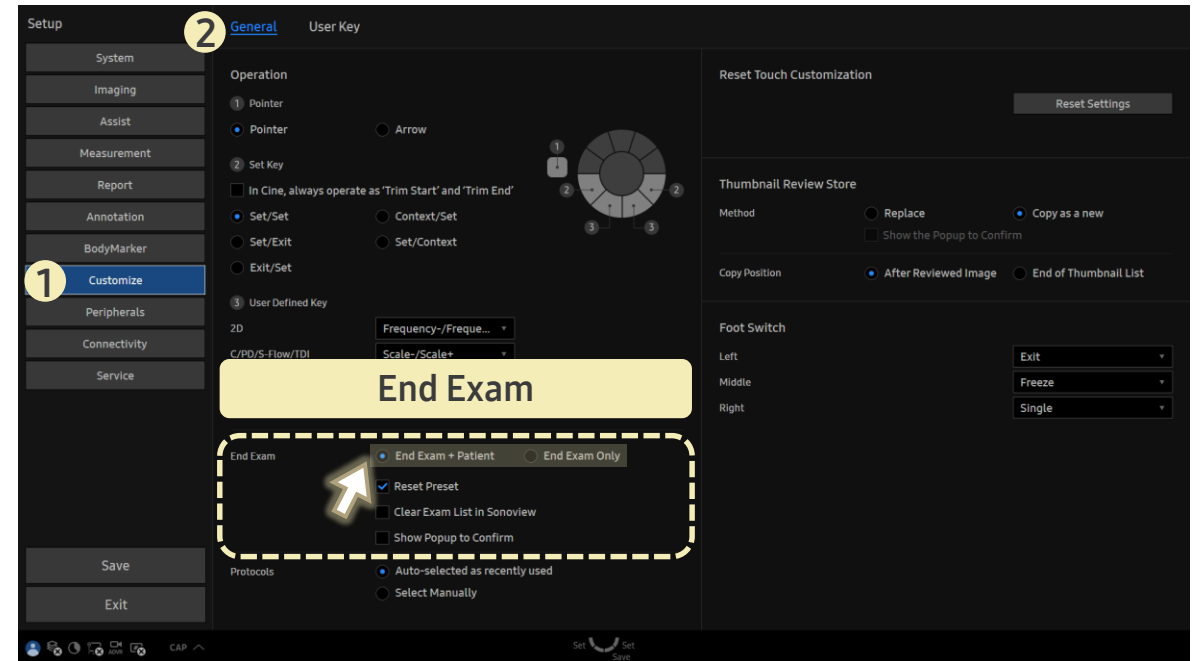

If needed, you can configure the following **End Exam** settings.

- **Reset Preset:** Resets the preset when **End Exam** is selected.
- **Show Popup to Confirm:** Enables or disables a confirmation popup when **End Exam** is selected.