


FAQ

How to Adjust Trackball Speed

[FAQ – Trackball Speed]

- Question
Can I customize the trackball speed depending on the type of function I'm using?
- Answer
 - Path: Setup > System > General
 - 1. Go to **Trackball Speed**.
 - 2. Adjust the speed for each category from **Slow** to **Fast**.
 - ▶ In **Scan Mode**, the setting affects tools such as the Pointer, M/Doppler Cursor, and 3D/4D ROI.

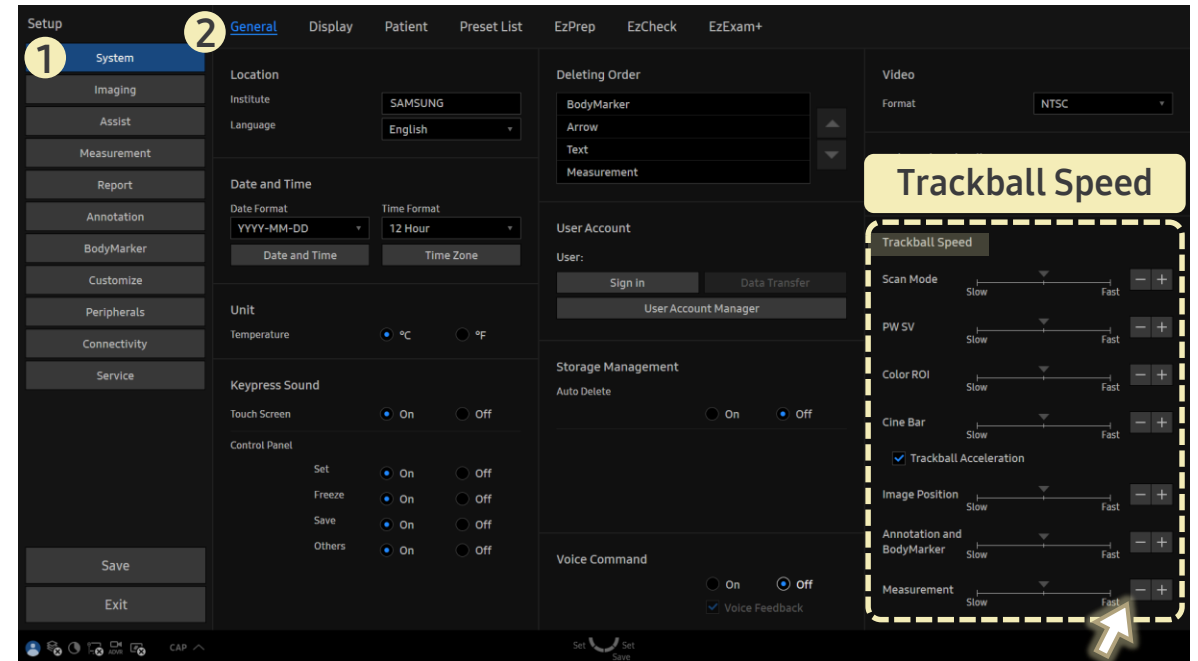

Trackball Acceleration

- For **Cine bar**, you can turn **Trackball Acceleration** on or off by checking the checkbox.