

FAQ

How to Activate BiometryAssist™


[FAQ – BiometryAssist™]

- Question
I want to perform Fetal Biometry, but **Assist** button is not visible on the touchscreen. (**Assist** button allows automatic measurement by simply selecting a measurement item.)

- Answer

■ Path: Setup > Assist > OB > General

Activate **BiometryAssist™** by clicking **On**.

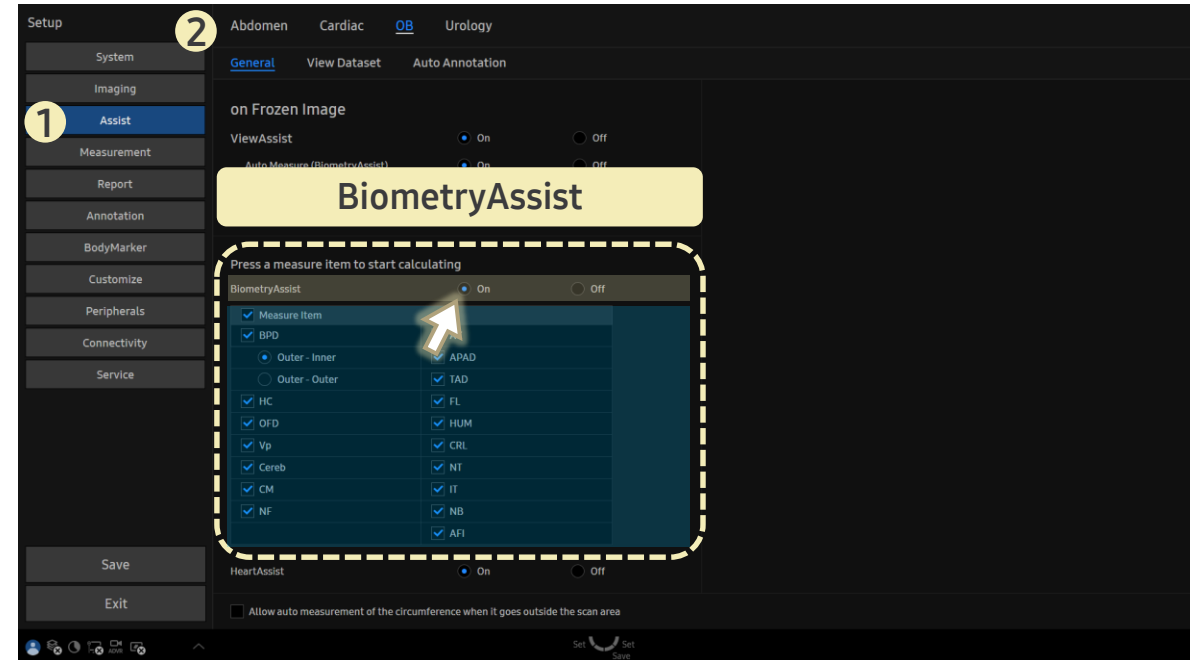

- You can selectively choose the measurement items to use with **BiometryAssist™** by selecting them from the list.
- Measurement items on the touchscreen that support **Assist** are labeled with "**Assist**".