

FAQ

OB Application

How to Activate Auto Measurement by Pressing the Measure Key

[FAQ – Auto Measure in OB]

- Question
Can I start **AutoMeasure** directly by pressing the **Measure** key on the control panel, without tapping the **AutoMeasure** button on the touchscreen?

- Answer

■ Path: Setup > Assist > OB > General

1. Make sure **ViewAssist** is set to **On**.
2. Check the box for **Start AutoMeasure by Pressing Measure Key**.



Start Auto Measure by Pressing Measure Key

- When checked: **AutoMeasure** starts by pressing the **Measure** key on the control panel.
- When unchecked: Use the **AutoMeasure** button on the touchscreen to start.