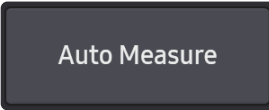



FAQ

OB Application

**Why is the Auto Measurement Button
not Visible on the Touchscreen**

[FAQ – Auto Measure in OB]

- Question
I don't see the **Auto Measurement** button on the touchscreen. What should I check?
- Answer
 - ▶ The button appears in **OB Application** in **Freeze** mode (including image review) and **Measure** modes.
 - Go to **Utility > Touch Customization** and check if the **Auto Measure** button is assigned. 
 - Go to **Setup > Assist > OB > General** and make sure **ViewAssist** is set to **On**.

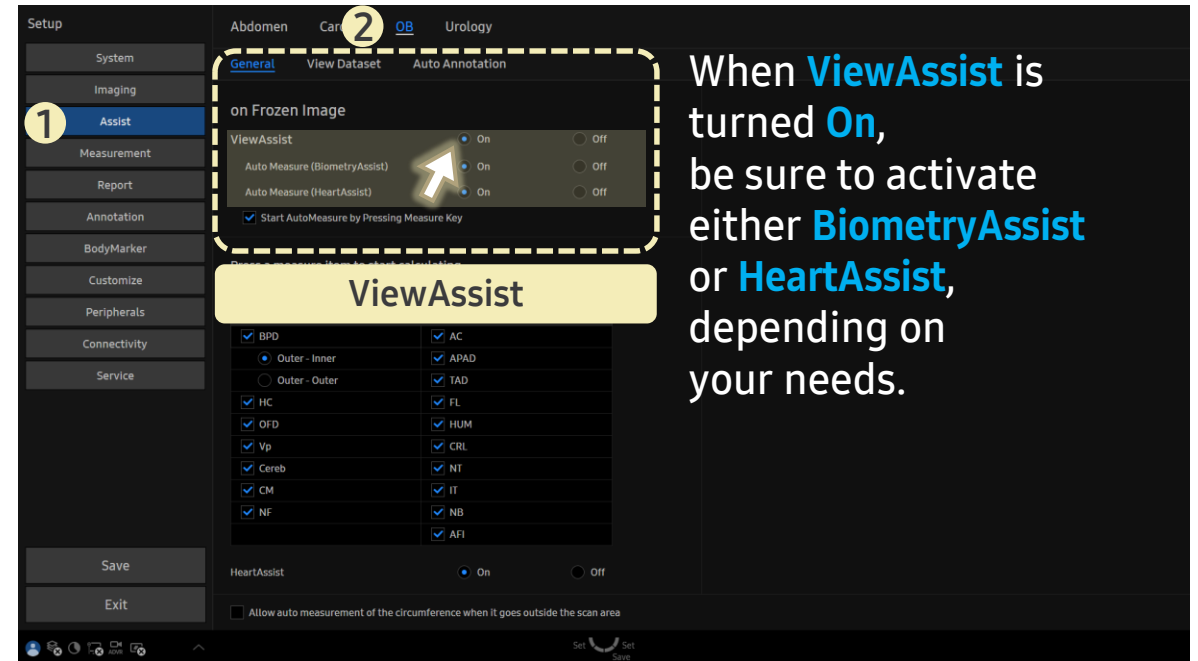

When **ViewAssist** is turned **On**, be sure to activate either **BiometryAssist** or **HeartAssist**, depending on your needs.

- To configure which views and measurement items are used for **Auto Measurement**, go to
- **Setup > Assist > OB > View Dataset.**