


# FAQ

## How to Make Automatic Kidney Measurement

## [FAQ – AbdomenAssist™]

- Question  
The **Assist** button is active when scanning the kidney, but measurements still need to be made manually. Why is that?


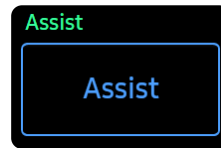


- Answer


■ Path: Setup > Assist > Abdomen

Make sure **AbdomenAssist** is set to **On**.

- ▶ If **Off** is selected, **Assist** functions will not work automatically, even if the button is active.

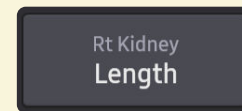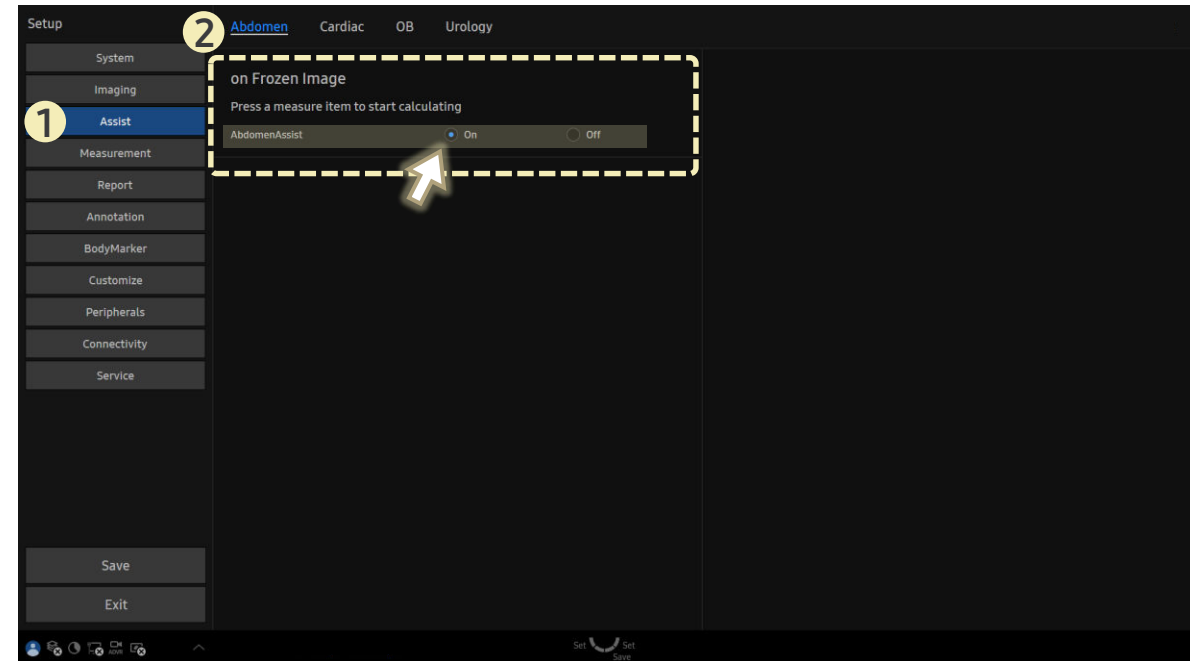

**Assist not applied**  
(AbdomenAssist: Off)

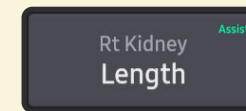

**Assist active**  
(AbdomenAssist: On)